

LIGHTLas SLT Deux-V

SLT/YAG COMBINATION SYSTEM
with Vitreolysis



QUICK START GUIDE

LIGHTLas SLT Deux-V: QUICK START GUIDE



Setting Up

Step 1

Turn the Table ON

Lasers are powered through the table. Make sure you turn table on first with the on-switch.

Step 2

Adjust Table Height

Make sure the green light is ON. Adjust the height using the UP and DOWN arrows.

Step 3

Unlock the Emergency Switch

Turn switch counter-clockwise until it pops out. Nothing will turn on until switch is unlocked.

Step 4

Turn on Slit Lamp

Turn the key to turn on the Slit Lamp.

Step 5

Press Button that says Laser Power

This turns on YAG and SLT. Machine will self calibrate.

Step 6

The Boost

Press for sudden boost of illumination during treatment.

Step 7


Illumination Intensity

Turn knob to gradually adjust the intensity of illumination.

Step 8

Adjust Arm Rest Height

Turn black knob to loosen hold on arm rest. Adjust height for comfort and tighten. Move arm rest to opposite side using the hole in table, or remove arm rest entirely.

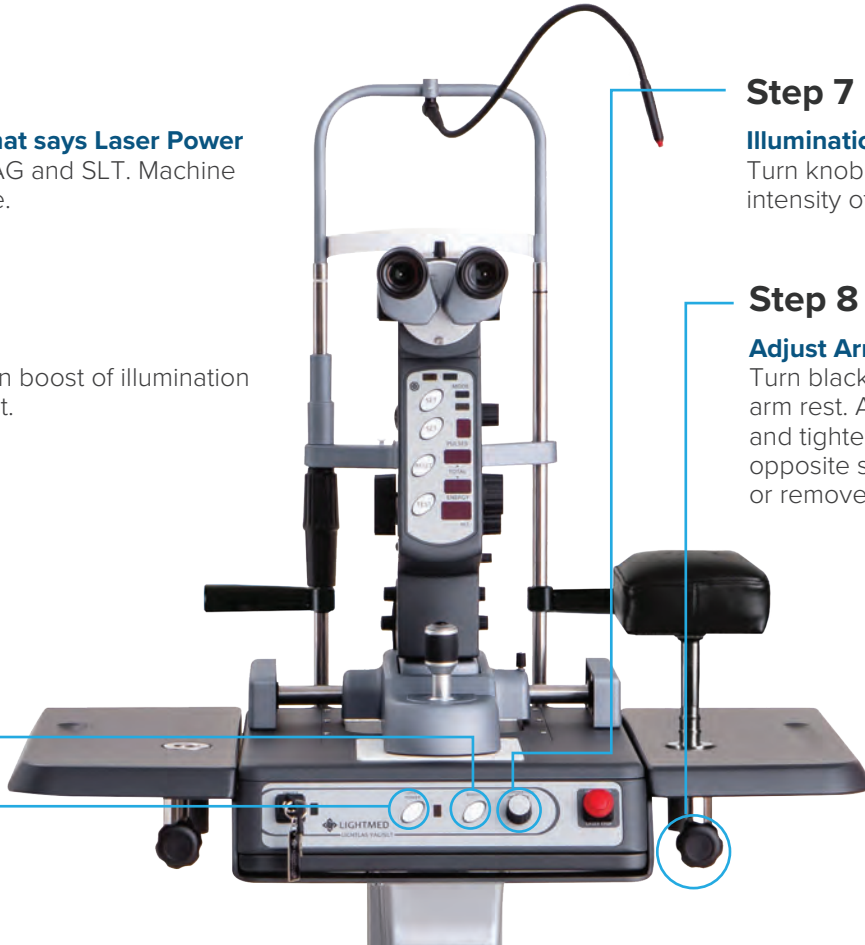

YAG & SLT Laser Settings

- 9 Magnification Changer**
Turn knob to desired magnification.
- 10 Switch from YAG to SLT Mode**
- 11 Offset (YAG mode only)**
Turn dial to adjust the offset. Make sure the dial says + FOR POSTERIOR OFFSET, or - FOR ANTERIOR OFFSET. This ranges from -500 to +500.
- 12 12a - YAG Energy Setting**
Turn to adjust for YAG energy setting.
12b - SLT Energy Setting
Turn to adjust for SLT energy setting.
- 13 Energy Reading**
View your adjusted parameters here.
- 14 Test Button**
Fires the laser while keeping the shutter closed. Safest way to check laser output energy.
- 15 Set Button**
Changes the laser from Standby to Treat mode. Push once desired parameters are set.
- 16 Set Pulse**
Change pulses from 1, 2, or 3 pulses per laser shot.
- 17 Total Counter**
Top shows total number of shots. Bottom shows total energy used. Record values after procedure.
- 18 Reset**
Resets the counter.

